

Within a motor, strength and balance re-education programme, these tilt tables help to gradually reach an erect posture from a static prone or supine position when patients are unable to load their own body weight on their lower limbs. Chinesport's static tables in a supine or prone position are recommended for use for both adults and children. They have a three year guarantee.

#### PRODUCT CODE MEANING

The product code for Chinesport static tables depends on the configuration of the various options available. Each position in the code has a specific meaning, and corresponds to one or more characteristics of the product. Soon we propose introducing what is termed an "open code" with a number of options to be chosen freely, as already offered for our examination and therapy couches, or our shower stretchers.

Specifically, a model with gas spring servo-assisted tilt adjustment can be chosen, or an electrical system powered by mains power or a rechargeable battery. Electrical height adjustment is also provided for some models of the table, if you also want alternative use for physical therapy sessions.

The drives offered for tipping the table upright, provide for working safely with various loads, up to 140 kg or 180 kg. Some icons are shown for each model presented, for easier identification of the properties described.



#### The FIRST PART

of the code is predefined, and identifies the principal characteristics.

#### The SECOND PART

of the code relates to the colour choice for the upholstery.

	PREDEFINED											
•	1	2	3	4	5	6	7	8				

POSITION	VARIABLE CODE MEANING	VALUES AVAILABLE						
1-2		SET VALUES						
		1 - no height adjustment						
3	TYPE OF ADJUSTMENT	2 - electrical height adjustment						
		3 - gas spring tilt adjustment						
		E - mains power supply						
4	TYPE OF POWER SUPPLY	B - rechargeable battery power supply						
		<b>G</b> - no electrical supply						
5	SAFETY WORKING LOAD	<b>1</b> - up to 140 kg						
J	OAI ETT WOTINING LOAD	<b>2</b> - up to 180 kg						
		1 - 3 sections, without head section for static supine position						
		' '						
6	NUMBER OF SECTIONS	2 - 4 sections, with head section for static supine position						
		3 - 1 section, for static supine position						
		4 - 4 sections, for static supine position						
7	TYPE OF IMITATION LEATHER	W - GIOTTO collection, rounded edges						
		X - GIOTTO collection, edges with band						
8	COLOUR	24 colours available						



#### POSITION 8

## 1 2 3 4 5 6 7 8

#### ? Colour of upholstery for the sections.

When ordering remember to state the colour code for the upholstery. Available on a standard range of 24 colours.



#### XCAT0020131

**FOLDER OF UPHOLSTERY COLOURS** 

For designers as well as for the single professional, who would like to verify the real colour of the imitation leather before taking a purchase decision, we can send a samples folder for free.





## TiltTables Accessories

#### STRAPS, SUPPORTS AND UPHOLSTERED ITEMS

Chinesport's static tables are designed as "open products", that is, the professional is able to choose the accessories they require to complete the unit. A series of important accessories are presented here that are used to complete the static table's configuration, and provide the best answer to the specific needs of operators and users.



The model shown is code LT1E14W6 - TILT TABLE E PRO with accessory AC0310 for static supine use.



ACO005 BASIC STRAP
Provides better restraint for the patient; can be fitted at the desired height by means of special guides. (single piece)



AC0560 PADDED TRUNK STRAP
A strap with a padded support for comfortable, safe, retention of the person. It is recommended for fitting at the trunk or pelvis. The strap's adjustment and fixing system is easy for the operator to use. It can be used for all static tables, in the supine position. (single piece)



ACOO88 PADDED LEG STRAP

A broad strap with a padded support for containing both lower limbs. It has a double fixing on the side and a system that facilitates fitting by the operator. It can be used for all static tables, in the supine position. (single piece)



**ACOOO6 STRAP PADDING**Padded cushion offering greater and more comfortable support to the patient. To be used in combination with strap code ACOOO5. (single piece)



AC0561 SEPARATE LEG STRAPS
A strap with a padded support for positioning and stabilising an individual lower limb. This type of strap can be used for static tables that have a central guide for anchoring the strap. In particular, it can be used for all static tables, in the supine position. (2 pieces)



ACO087.W ? CERVICAL CUSHION

A padded, shaped cushion for lateral containment and stabilisation of the neck. This cushion has a special strap that can be fitted or removed quickly, depending on the specific situation. (single piece)

? CHOICE OF UPHOLSTERY COLOR - important: always specify the upholstery code along with the chosen item when this option is available.

















































#### STRAPS, SUPPORTS AND UPHOLSTERED ITEMS



**ACOOO3 LATERAL SUPPORTS**Side supports made from soft material, adjustable in height, width and depth. (2 pieces)



ACO517.W ? PADDED SUPPORTS
Shaped side supports that offer a larger contact surface with the patient. These are supplementary or alternative to article code AC0003 and can be adjusted in height, width, and depth. (2 pieces).



**ACOOO4 HAND GRIPS**Side handles to allow the patient to grip and support themselves while exercising in a static supine position.

They can be adjusted in height and doubt.

themselves while exercising in a static supine position. They can be adjusted in height and depth. (2 pieces).



**AC0090 FOREARM SUPPORTS** 

Forearm supports with hand grips for a comfortable hold. They can be adjusted in height and rotated, while the hand grips also have a depth adjustment. (2 pieces).



**AC0002** INTRA-EXTRA SANDALS

Once fitted to the footplates, they restrain and stabilise the feet. Can be adjusted into intra-extra rotation and anteroposterior rotation. (2 pieces)



AC0020.W ? BREATHING HOLE PLUG

Chinesport's static tables come with a standard breathing hole and plug. It can be ordered as an accessory in the same colour as the table.




#### SEPARATE FOOTREST FLEXO-EXTENSION

#### **AC0096 WEDGE FOR CLUB FEET**

The foot rests can be adapted to provide a greater incline of the footrest that can go to  $-30^{\circ}$  for extension and  $+10^{\circ}$  for flexion, where required for patients with a club foot. The specific accessory must therefore be ordered.

 $+10^{\circ}/-30^{\circ}$ 

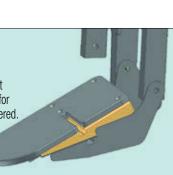

The model shown is code LT1E14W6

- TILT TABLE E PRO with accessory ACO309 Service table.

## TiltTables Accessories

#### **ELECTRICAL ACCESSORIES**

Should the standard battery provided be damaged or wear down, an additional battery can be ordered using the accessory code shown below. It is very important to respect the battery recharging procedure in order to extend its working life. All static tables that are battery powered, come with a battery charger as standard equipment. However, a spare or replacement battery charger can be ordered. Also, all models with electrical tilt adjustment, come with a standard handset with magnets. However, you can also include an additional handset or foot switch, with a support in the frame.



#### **AC0319 BATTERY CHARGER**

This can be used to recharge the battery supplied or spare batteries, by connecting it to the electricity mains. The power supplies that can be used are 110/220V - 50/60 Hz - 500 mA. It comes with a standard Italian plug, compatible with SHUKO socket. By request, it can be supplied with other type of plug.



#### **AC0318 WALL-MOUNTED BATTERY**

**CHARGER** This can be used to recharge the battery supplied or spare batteries, by connecting it to the electricity mains. The power supplies that can be used are 110/220V - 50/60 Hz - 500 mA. It comes with a standard SHUKO plug. By request a cable can be provided with another plug.



#### **01132 SPARE BATTERY**

Allows continuous use, while the other battery is charging. The internal power supply is 24 V / max 10 AT.



#### **AC0039** ADDITIONAL HANDSET

Ergonomic, new generation handset with magnets - can be attached to the frame of the table as the operator wishes. It comes with power cord.



#### **AC0575 PEDAL SUPPORT**

Can be fitted on either side of the bed, as the operator wishes, and is used to house accessory AC0083.



#### **AC0083** ADDITIONAL CONTROL

Can be used as a foot switch, along with the relevant support, code AC0575, or as a hand-held control. It is fitted with magnets to make it easy to attach to the frame wherever you wish.



#### **AC0578** SAFETY DEVICE

This device disables all available electrical functions when the carer is away.



#### TILT INDICATOR

All Chinesport's static tables have a protractor that is a standard fitting, to indicate the tilt in degrees. During the rehabilitation programme, it is therefore possible to monitor the progression in reaching and maintaining an upright position.

#### **SPECIFIC ACCESSORIES**



#### **ACOO81** SUPINE STATIC TABLE

This work table can be used with one-, three- or foursection tables when the patient is in a static supine position. The table is contoured so forearms can be easily rested on it. Can be adjusted in height, depth and tilt. Dimensions: 700 x 560 mm



#### **AC0309 PRONE STATIC TABLE**

This work table can only be used with PRONO tilt tables, ref. models LT1B14 and LT1E14. Can be adjusted in height, depth and tilt. Dimensions: 700 x 560 mm



#### ACO308 BOBATH TILT TABLE

This work table can only be used with BOBATH tilt table, ref. model LB492F2. The table is contoured to provide comfortable support to forearms. Can be adjusted in height, depth and tilt. Dimensions: 700 x 560 mm



**ACO310.W ? SUPINE STATIC HEAD SECTION** This part can be used as an alternative when the tilt table is in supine position. The accessory is only suitable for models LT1B14 and LT1E14. It is provided with guides to allow for restraining straps to be fitted. Dimensions: 680 x 600 mm



#### AC0311 ABDUCTION BLOCK

Helps keep lower limbs separated when the patient is in a static position. It can be fitted whenever required and is height adjustable. Only for models LT1E14, LT1B14, LB492F2.



#### **ACO312** GUIDE FOR BASE STRAP

Helps to safely fit the lower limbs restraining straps. If required, a separate fitting device can be used. Only for models LT1E14, LT1B14, LB492F2.

**? CHOICE OF UPHOLSTERY COLOR** - important: always specify the upholstery code along with the chosen item when this option is available.

## TiltTables for supine position



## L T 3 G 3 3 W ? TILT TABLE G

footplate is ready to fit the sandals AC0002.






Single-section tilt table for static exercises in supine position for up to 120 kg. Up to 90° tilt via two-gas spring servo-assisted system. Lifting takes place in a continuous movement and with minimal effort for the carer, while the user is on board. It is fitted with tilt angle indicator; height from ground is 85 cm. The table does not come with any containment band, which must be chosen from among the accessories available. It can be moved with a mobile lift or medical equipment trolley, clearance under the table base is 17 cm. Transport is made easier by the wide diameter, braked casters, one of them being directional. The



Clearance under the table allows access by patient lifts or medical equipment trolleys

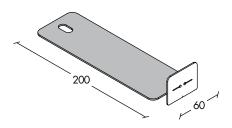







TECHNICAL SPECIFICA	TIONS						
ARTICLE CODE	LT3G33						
	Tilt table G						
Number of sections	1						
Maximum safe working load (kg)	120						
Tilt adjustment	with gas spring						
Power supply	not applicable						
Tilt indicator in degrees	standard						
Table height from the floor (cm)	85						
Maximum user height (cm)	200						
Caster diameter (mm)	125						
Individual brake system	on each single wheel						
Steering wheel	standard						
Padding upholstery safety class	1 IM						
Padding thickness (mm) / density (kg/m3)	30 / 30						
Weight (kg)	60						

#### **ACCESSORIES**

**AC0005 BASIC STRAP AC0006 STRAP PADDING** AC0560 PADDED TRUNK STRAP **AC0088 PADDED LEG STRAP** AC0087.W ? CERVICAL CUSHION **ACOOO3 LATERAL SUPPORTS** AC0517.W ? PADDED SUPPORTS **AC0004 HAND GRIPS** 

AC0090 FOREARM SUPPORTS **AC0002 INTRA-EXTRA SANDALS** 

AC0020.W ? BREATHING HOLE PLUG

**AC0081 SUPINE STATIC TABLE** 

?	CHOICE OF LIPHOLSTERY COLOR	- important: always specify the upholstery code along with the chosen item when this ontion is available.
-	CHOICE OF UPHOLSTERT COLOR	- important, simans specify the hodorstern code stodo mittine chosed tieth maeu fuls option is systistile

Α	N		7	K	S			Т	1	6	E	Z	G	F	Н	9	Q	R	2	3	L	M	P
---	---	--	---	---	---	--	--	---	---	---	---	---	---	---	---	---	---	---	---	---	---	---	---

## TiltTables for supine position



L T 1 E 1 1 W 2 TILT TABLE E





Three-section tilt table for static exercises in supine position for up to 140 kg. Up to 90° tilt with wall power supply, 24V low voltage, and operated by hand control. It is fitted with tilt angle indicator; height from ground is 56 cm. The table does not come with any containment band, which must be chosen from among the accessories available. A central guide between the two lower limbs support sections allows for the restraining strap to be better fitted and secured. Transport is made easier by the wide diameter, braked casters, one of them being directional. Footplates can be independently adjusted by +/- 20°.

#### 1 | B | 1 | 1 | W | TILT TABLE B





Version with 24 V rechargeable battery supply. Battery charger included.

### T | 1 | E | 2 | 1 | W ? TILT TABLE E PLUS






Version with wall power supply for up to 180 kg. An additional manual lowering system is provided, in case a black-out occurs.

#### 1 B 2 1 W ? TILT TABLE B PLUS





Version with 24 V rechargeable battery supply for up to 180 kg. Battery charger included.

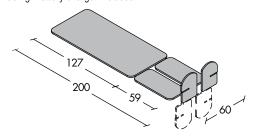



TECHNICAL SPECIFIC	ATIONS					
ARTICLE CODE		LT1B11	LT1E21	LT1B21		
	Tilt table E	Tilt table B	Tilt table E Plus	Tilt table B Plus		
Number of sections	3	3	3	3		
Maximum safe working load (kg)	140	140	180	180		
Tilt adjustment	electrical	electrical	electrical	electrical		
Power supply	mains	battery	mains	battery		
Tilt indicator in degrees	standard	standard	standard	standard		
Table height from the floor (cm)	56	56	56	56		
Maximum user height (cm)	200	200	200	200		
Caster diameter (mm)	125	125	125	125		
Individual brake system	on each single wheel					
Steering wheel	standard	standard	standard	standard		
Padding upholstery safety class	1 IM	1 IM	1 IM	1 IM		
Padding thickness (mm) / density (kg/m3)	30 / 30	30 / 30	30 / 30	30 / 30		
Weight (kg)	65	65	65	65		

For models that use mains power: 220 V 50 / 60 Hz / 24 V 70 VA; For models that use battery power: 24 V / max 10AT; Battery charger: 110 / 220 V - 50 / 60 Hz - 500 mA.

#### **ACCESSORIES**

**AC0005 BASIC STRAP** 

**AC0006 BAND PADDING** 

AC0560 PADDED TRUNK BAND

**AC0561 SEPARATE LEG BANDS** 

**AC0088 PADDED LEG BAND** 

AC0087.W ? CERVICAL CUSHION

**AC0003** LATERAL SUPPORTS

AC0517.W ? PADDED SUPPORTS

**AC0004 HAND GRIPS** 

**AC0090 FOREARM SUPPORTS** 

**AC0002 INTRA-EXTRA SANDALS** 

**AC0096 WEDGE FOR CLUB FEET** 

**AC0319 BATTERY CHARGER** 

**AC0318 WALL-MOUNTED CHARGER** 

01132 SPARE BATTERY

**AC0039 ADDITIONAL HANDSET** 

**AC0575 PEDAL SUPPORT** 

**AC0083 ADDITIONAL CONTROL** 

**AC0578 SAFETY DEVICE** 

**AC0081 SUPINE STATIC TABLE** 

CHOICE OF UPHOLSTERY COLOR - important: always specify the upholstery code along with the chosen item when this option is available.

























































## TiltTables for supine position



L T 1 E 1 2 W ? TILT TABLE ET

140Kg



Four-section tilt table for static exercises in supine position for up to 140 kg. Up to 90° tilt with wall power supply, 24V low voltage, and operated by hand control. It is fitted with tilt angle indicator; height from ground is 56 cm. Head end section can be adjusted up/down by gas spring. The table does not come with any containment band, which must be chosen from among the accessories available. A central guide between the two lower limbs support sections allows for the restraining strap to be better fitted and secured. Transport is made easier by the wide diameter, braked casters, one of them being directional. Footplates can be independently adjusted by +/- 20°.

L T 1 B 1 2 W TILT TABLE BT





Version with 24 V rechargeable battery supply. Battery charger included.

L T 1 E 2 2 W ? TILT TABLE ET PLUS





Version with wall power supply for up to 180 kg.

An additional manual lowering system is provided, in case a black-out occurs.

L T 1 B 2 2 W 2 TILT TABLE BT PLUS



Version with 24 V rechargeable battery supply for up to 180 kg. Battery charger included.



CE

TECHNICAL DATA	4					
ARTICLE CODE		LT1B12	LT1E22	LT1B22		
	Tilt table E	Tilt table B	Tilt table E Plus	Tilt table B Plus		
Number of sections	4	4	4	4		
Maximum safe working load (kg)	140	140	180	180		
Tilt adjustment	electrical	electrical	electrical	electrical		
Power supply	Mains	Battery	Mains	Battery		
Tilt indicator in degrees	standard	standard	standard	standard		
Table height from the floor (cm)	56	56	56	56		
Maximum user height (cm)	200	200	200	200		
Caster diameter (mm)	125	125	125	125		
Individual brake system	on each single wheel					
Steering wheel	standard	standard	standard	standard		
Padding upholstery safety class	1 IM	1 IM	1 IM	1 IM		
Padding thickness (mm) / density (kg/m3)	30 / 30	30 / 30	30 / 30	30 / 30		
Weight (Kg)	65	65	65	65		

For models that use mains power:  $220 \ V \ 50 \ / \ 60 \ Hz \ / \ 24 \ V \ 70 \ VA$ ; For models that use battery power:  $24 \ V \ / \ max \ 10AT$  Battery charger:  $110 \ / \ 220 \ V \ - \ 50 \ / \ 60 \ Hz \ - \ 500 \ mA$ .

#### **ACCESSORIES**

ACOOO5 BASIC STRAP

AC0006 BAND PADDING

AC0560 PADDED TRUNK BAND

AC0561 SEPARATE LEG BANDS

AC0088 PADDED LEG BAND

AC0087.W ? CERVICAL CUSHION

ACOOO3 SUPPORTS

AC0517.W ? PADDED SUPPORTS

**AC0004 HAND GRIPS** 

AC0090 FOREARM SUPPORTS

**AC0002 INTRA-EXTRA SANDALS** 

**AC0096 WEDGE FOR CLUB FEET** 

AC0020.W ? BREATHING HOLE PLUG

**AC0319 BATTERY CHARGER** 

AC0318 WALL-MOUNTED CHARGER

01132 SPARE BATTERY

AC0039 ADDITIONAL HANDSET

AC0575 PEDAL SUPPORT

**AC0083** ADDITIONAL CONTROL

**AC0578 SAFETY DEVICE** 

AC0081 SUPINE STATIC TABLE

**?** CHOICE OF UPHOLSTERY COLOR - important: always specify the upholstery code along with the chosen item when this option is available.

A N 8 7 K S B 4 T 1 6 E Z G F H 9 Q R 2 3

## **TiltTables** for prone and supine position







	LT1B14			
Tilt table E	Tilt table B			
4	4			
140	140			
electrical	electrical			
Mains	Battery			
standard	standard			
56	56			
190	190			
125	125			
on each single wheel	on each single wheel			
standard	standard			
1 IM	1 IM			
30 / 30	30 / 30			
65	65			
1 E N S 5 1 1 C S 1	LT1E14 Tilt table E  4 40 electrical Mains standard 66 90 125 on each single wheel standard IM 80 / 30			

For models that use mains power: 220 V 50 / 60 Hz / 24 V 70 VA; For models that use battery power: 24 V / max 10 AT Battery charger: 110 / 220 V - 50 / 60 Hz - 500 mA.

AC0005 BASIC STRAP AC0006 BAND PADDING

AC0088 PADDED LEG BAND

ACOOOS TABBES EEO BARTS
ACOOOS LATERAL SUPPORTS

AC0517.W ? PADDED SUPPORTS

**AC0004 HAND GRIPS** 

AC0090 FOREARM SUPPORTS

**AC0002 INTRA-EXTRA SANDALS** 

**AC0319 BATTERY CHARGER** 

**AC0318 WALL-MOUNTED CHARGER** 

01132 SPARE BATTERY

AC0039 ADDITIONAL HANDSET

**AC0575 PEDAL SUPPORT** 

**AC0083** ADDITIONAL CONTROL

**AC0578 SAFETY DEVICE** 

AC0309 PRONE-SUPINE STATIC TABLE

AC0310.W ? SUPINE STATIC HEAD SECTION

AC0311 ABDUCTION BLOCK
AC0312 GUIDE FOR BASE BAND

? CHOICE OF UPHOLSTERY COLOR - important: always specify the upholstery code along with the chosen item when this option is available.

1									-		-	-										
	Α	N	7	K	s		T	1	6	E	Z	G	F	н	9	Q	R	2	3	L	М	P












Fitting the "head end section for use in supine static exercises" - Accessory AC0310



Height-adjustable "Abduction block" – Accessory ACO311



"Abduction block" can be easily slotted into the central arrangement

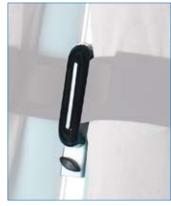

Lower limbs restraint with "strap guide" – Accessory ACO312

## **Bobath Table** with tilt function



#### L B 4 9 2 F 2 X 2 TILT TABLE BOBATH 2ET

Two-section Bobath table for treatment and static exercises in supine position. Height-adjustable up to 250 kg (safe load), or tilt adjustable from 0° to 90° up to 190 kg, with wall power supply, 24V low voltage, and operated by hand control. It is fitted with tilt angle indicator; minimum height from ground is 55 cm. The table does not come with any containment band, which must be chosen from among the accessories available. Head end section can be adjusted up/down by gas spring; it is also provided with breathing hole plug. A central arrangement between the lower limbs support sections also allows for the restraining strap to be better fitted and secured by means of accessory AC0312. Leg separator accessory AC0311 can also be fitted. Table padding with side panel. Casters have a pedal-activated braking system. Footplates can be independently adjusted by +/- 20° and removed when required during physical therapy sessions.















TECHNICAL DATA									
ARTICLE CODE	LB492F2X								
	Tilt table Bobath 2ET								
Number of sections	6								
Maximum safe working load - raised (kg)	250								
Maximum safe working load - static (kg)	190								
Tilt adjustment	electrical								
Height adjustment (cm)	55 - 105								
Power supply	Mains								
Tilt indicator in degrees	standard								
Maximum user height (cm)	200								
Caster diameter (mm)	65								
Brake system	centralized								
Steering wheel	not applicable								
Padding upholstery safety class	1 IM								
Padding thickness (mm) / density (kg/m3)	40 / 30								
Weight (kg)	170								

#### **ACCESSORIES**

ACOOOS BASIC STRAP

AC0006 BAND PADDING
AC0003 LATERAL SUPPORTS

AC0004 HAND GRIPS

**AC0002** INTRA-EXTRA SANDALS

AC0096 WEDGE FOR CLUB FEET

AC0578 SAFETY DEVICE

AC0308 BOBATH TILT TABLE AC0311 ABDUCTION BLOCK AC0312 GUIDE FOR BASE BAND

For models that use mains power: 220 V 50 / 60 Hz / 24 V 70 VA; For models that use battery power: 24 V / max 10AT Battery charger: 110 / 220 V - 50 / 60 Hz - 500 mA.

? CHOICE OF UPHOLSTERY COLOR - important: always specify the upholstery code along with the chosen item when this option is available.

135 200

A N 8 7 K S B 4 T 1 6 E Z G F H 9 Q R 2 3 L M P

# ILT TABLES

## **Bobath Table** with tilt function

1) Note: A specific safety device is fitted to allow for an alternative way of adjusting the height or tilt.

2) Note: A specific safety device is fitted to allow for the table to be lowered manually in case of power interruption.



Note: This model was designed to help both carer and patient during gym rehabilitation programmes, to avoid the time and possible inconvenience of a transfer from a specific table to another.



Fitting the footplate assembly for use with static exercises in supine position



Removable footplate assembly


#### SOME ACCESSORY APPLICATION EXAMPLES



Foot restraining sandals accessory AC0002 (2 pieces)




Strap guide accessory AC0312 (single piece)

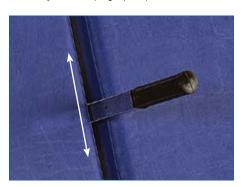


Forearm supports accessory AC0004 (2 pieces)



Abduction block accessory for Leg separation. ACO311 (single piece)

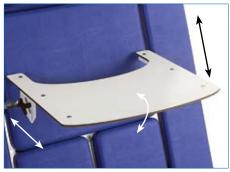


Work table for static exercises in supine position accessory AC0308 (single piece)

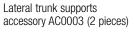

## Accessories Cushions for posture

#### **CUSHIONS FOR POSTURE**


Postural cushions are available in various shapes and sizes to meet the professional's varied requirements to position a patient correctly. Our cushions are made from variable density foam and have a synthetic leather cover which is fire-resistant and anti-bacterial, easy to clean and durable. Standard density is 21 kg/m<sup>3</sup>. Cushion covers have a zip for easy removal, and come in a choice of upholstery from an assortment of 24 color options. Cushions can be purchased in the same type of upholstery as the examination or therapy table.

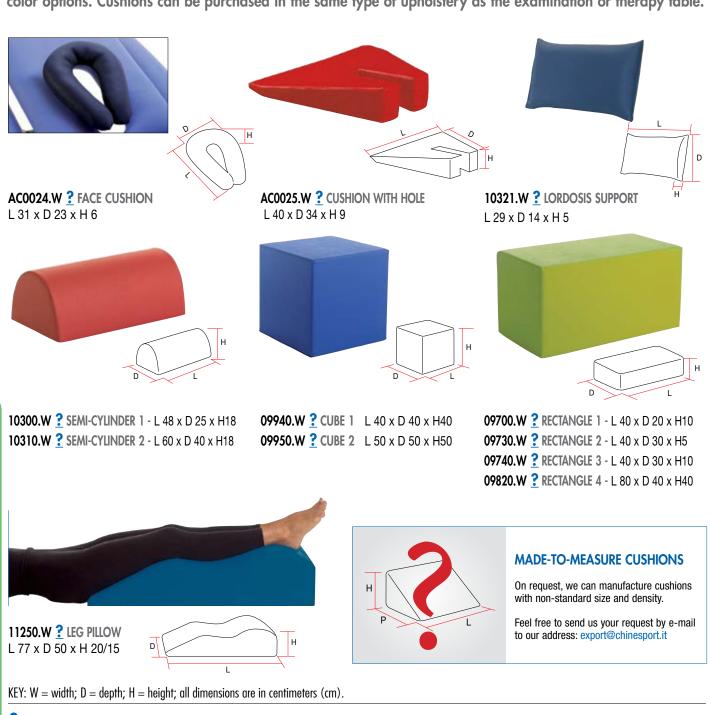


CHOICE OF UPHOLSTERY COLOR - important: always specify the upholstery code along with the chosen item when this option is available.

Z
G
F
H
9
Q
R
2
3
L
M
P

## Accessories Cushions for posture

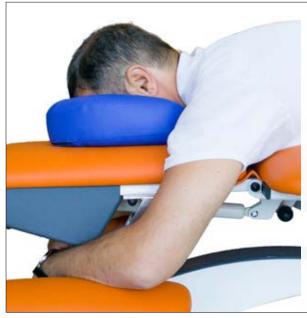

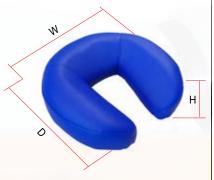



AC0881.W ? FACE CUSHION 2 A cushion with a special shape and density which makes comfortable the prone position of the patient during treatment and therapies. This cushion is advisable for offering great relaxation to patients. Dimensions: W 29 x D 31 x H 9 cm.


#### **REMOVABLE COVERS**

All Chinesport mattresses and cushions have a synthetic leather cover which is fire-resistant and anti-bacterial, highly durable, removable and washable.



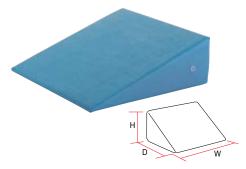

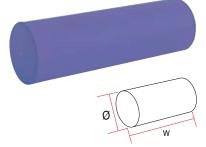

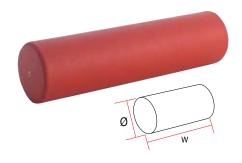

**09600.W** ? WEDGE 1 - W 25 x D 25 x H 10 **09610.W** ? WEDGE 2 - W 35 x D 35 x H 10 **09620.W** ? WEDGE 3 - W 60 x D 45 x H 15 **09630.W ? WEDGE 4** - W 60 x D 45 x H 30 **09640.W** ? WEDGE 5 - W 60 x D 60 x H 15 **09650.W** ? WEDGE 6 -W 60 x D 60 x H 20

**10000.W ? CYLINDER** 1 - W 20 x Ø 8 10010.W ? CYLINDER 2 - W 35 x Ø 15 **10020.W** ? CYLINDER 3 - W 50 x Ø 15 **10030.W ? CYLINDER 4** - W 50 x Ø 25 10050.W ? CYLINDER 5 - W 100 x Ø 25

**10130.W** ? RIGID ROLL 1 - W 100 x Ø 30 10150.W ? RIGID ROLL 2 - W 100 x Ø 40 10160.W ? RIGID ROLL 3 - W 100 x Ø 50 **10200.W** ? RIGID ROLL 4 - W 50 x Ø 30



always return to their original shape. They offer an ideal support to the lumbar and lateral region, as well as to the upper and lower limbs. They are covered in a washable, breathable fabric.

> 01449 POSTURAL CUSHION 100 - ø cm 19 x 17 01450 POSTURAL CUSHION 110 - ø cm 19 x 31 01451 POSTURAL CUSHION 120 - ø cm 25 x 37

01452 POSTURAL CUSHION 8 - cm 40 x 20 x 13 H

01453 POSTURAL CUSHION 88 - cm 50 x 28 x 13 H

#### 01426 POSTURAL SET

This set includes 5 cushions, one for each type specified above, and a carry bag to store the items, size 33 x 33 x 56 h cm.

