## DOs

Wear compression therapy products only after consulting doctor or medical practitioner.

Handle with gentle care and save from prickles, sharp objects & ornaments; For better donning - use butlers & rubber gloves.

Store in cool & dry place at room temperature.

Wash your compression therapy products regularly in order to retain its functionality & hygiene.

Wash your products separately & delicately by hand using fine liquid detergents like - Ezee / Genteel / any shampoo (without moisturiser/ conditioner).

While washing use normal water at room temperature (not exceeding 25°-30°(C)).

After washing, dry the product on a horizontal surface & protect it from sunlight.

Wear the product as early as possible after getting up in the morning. Adjust silicone dotted bands several times in a day to avoid skin irritations & discomfort.

While walking use sports shoe for lower limb products. Ensure your footwear is in good condition for better life of products.

Orthopaedic products should be strictly used as per guidelines or as advised by Doctors.

Velcros of ortho braces should be fastened in sequence & in right individual channels only. Otherwise this may damage the product & other fabric.

## DON'Ts

Don't continue to wear any product, if pain or irritation persists. Consult your doctor or medical practitioner for further advice.

Don't use / wear any products with sharp finger nails, rings, rough / horny skin and cracked heel.

Don't expose any product to direct sunlight

Don't wash any product together with any other fabric and don't leave the velcro fasteners open to avoid damage to fabric.

Don't wash in washing machines & Don't use soap cake, detergent powder, chemicals, bleaching agents & fabric softeners or conditioners.

Don't dry clean or wash in hot water.

Don't hang the product after washing. Don't use dryers / heaters / iron.

Don't wear the product while bathing or at night while going to bed unless & until doctor has advised for the same.

Don't walk bare foot with lower limb - foot products and don't use footwear with defective inner lining otherwise there might be increased friction & can lead to damage of the product.

Don't make technical adjustments in your orthopaedic braces on your own in case of any problem.

Don't strap or unstrap the velcros randomly and avoid fastening beyond individual channel.

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